



TRIBEST DENTAL PRODUCTS CO., LTD

No.98 Zhongling Road, Sanmao Street, Yangzhong City, Jiangsu, China

Tel: +86-511-88091812

Fax: +86-511-88222305

Website: www.tribestdental.com

Email: kevin@tribest.cn

Instructions for Use

[Product Name]

Self-sealing sterilization pouch

[Instructions]

The pouches are intended for use as packing material for sterilization of items by steam, or ethylene oxide gas.

Select the compatible pouch according to your sterilization method and the size of the sterilized instrument.

Place item in pouch. The pouch should only be filled 3/4 of the packing volume to allow proper air evacuation and sterilant penetration. Carefully press the excess air out of the pouch and ensure there is at least 25mm of area available around all four sides of the enclosed items.

Peel off protective strip from adhesive band. Seal the pouch by folding the adhesive strip onto the pouch. Apply pressure from the center of the adhesive, towards the edges. Repeat this motion an additional time for extra security.

Place the pouch in the sterilizer. If using several pouches at once, when placing pouches in the sterilizer ensure plastic side of sterilization pouch always faces paper side of adjacent pouch to allow proper air evacuation and sterilant penetration. If possible use a pouch rack to eliminate stacking of the sterilization pouches.

Follow standard sterilization procedure. Check that pouches are dry upon removal.

Inspect package for damage, moisture, or any sign of potential contamination prior to opening and again after opening but before use of contents. Do not use contents if these conditions are present, as sterility could be compromised.

INSTRUCTIONS:

DO NOT DROP
items into
pouch



1. Insert items into unsealed pouch...
2. ...Peel off protective strip from adhesive band...
3. ...Fold flap on perforation and press down firmly



TRIBEST DENTAL PRODUCTS CO., LTD

No.98 Zhongling Road, Sanmao Street, Yangzhong City, Jiangsu, China

Tel: +86-511-88091812

Fax: +86-511-88222305

Website: www.tribestdental.com

Email: kevin@tribest.cn

[Caution]

Single-use only. Do not reuse.

Do not use if the pouch is damaged.

[Storage]

Keep away from light, heat and moisture. Store at temperature between 15 –35°C with a relative humidity of <70%.

It is recommended to use the sterilized instruments immediately after sterilization. If not used, please store in a cool, dry, ventilated, and non-corrosive gas environment.

[Shelf life]

It is recommended that the pouches are used within 3 years of manufacture.

The sterilization pouch maintains the enclosed devices sterile up to minimum 6 months after sterilization.