SDM-003(Rev.1 2015-12-08)

Instruction for Use

B&E Block-Dam

Light-Cured Bleaching Dam

Block-Dam is a light-reflecting resin barrier for isolating tissue adjacent to teeth being whitened. For single tooth bleaching, Block-Dam may be used to protect adjacent teeth.

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Made in Korea

B&E Block-Dam

Block-Dam is a light-reflecting resin barrier for isolating tissue adjacent to teeth being whitened.

For single tooth bleaching, Block-Dam may be used to protect adjacent teeth.

Intended use

Passively adhesive (sealing)

Application Method

1. Remove Cap from syringe
2. Place tip onto syringe.
3. Turn tip 90 degree to lock in place.
4. Apply the Block-Dam
5. Light curing for 20sec

Package

Block-Dam 1.2ml X 4syringe, Disposable tips

Cautions

1. Block-Dam only used for oral tissue.
2. Avoid contact with eyes.
3. In case of contact with the skin or eyes, clean with pure water several times, and then consult with a medical specialist if necessary.
4. Do not use expired Block-Dam
5. For dental use only.

Storage

1. Do not store it under conditions of high temperature or intense light.
2. Close the cap immediately after use.
3. Store in dry and cool place.
4. Warranty period is marked separately.

Side effects

In rare cases, Block-Dam may cause a sensitizing reaction for hypersensitive patients due to any of the ingredients of these materials. In these cases, Block-Dam must no longer be used.

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