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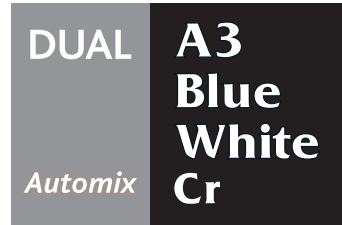
## Instruction for Use

# ChemiCore

**ChemiCore is dual-cured composite resin designed for the fabrication of all kinds of core build-ups & build-up fillings.**

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Manufacturer	
Authorized Representative In the European Community	
CE (Notified Body)	
Batch Code	
Date of Manufacture	
Use By	
Non Sterile	
Do not use if package is damaged	
Consult Instructions for use	
Temperature Limitation	
Keep dry	
Keep away from sunlight	
Caution	
Do not re-use	

**Manufacturer**  
**B&E KOREA Co., Ltd.**  
 #807, #802, B Dong, 60, Haan-ro,  
 Gwangmyeong-si, Gyeonggi-do, Korea  
 Tel. 82-70-7425-2818 Fax. 82-2-6112-2816

**EC Representative**  
**BIOGERI**  
 Am Pfeilshof 12, 22393 Hamburg, Germany  
 Tel. +49-40-4143-102150 Fax. +49-40-4143-102159

**B&E KOREA** ㈜비엔이코리아  
 www.bnekorea.com

Made in Korea

# ChemiCore

A3 · Blue · White · Cr

**ChemiCore is dual-cured composite resin designed for the fabrication of all kinds of core build-ups & build-up fillings. ChemiCore has 4 colors.**

**■ Indication for use**  
 Application of all build-up fillings

**■ Contraindication**  
 Allergic to any ingredient in ChemiCore

**■ Application method**

**1. Preparation before use**

- 1) Checkup expiry date and package of product before use.
- 2) Install eco-tip or mixing-tip to ChemiCore.

**2. How to use**

- 1) Wash, dry and prepare a tooth.
- 2) Prepare a cavity on the treatment region.
- 3) Perform pretreatment with etching and bonding (pay attention to the use of bonding)
- 4) Fill proper amount of ChemiCore to the treatment region.
- 5) Irradiate the light.

\*Auto-polymerization (in oral cavity): Within 5 min  
 (LED light curing unit: 20 sec, Halogen light curing unit: 40 sec)

6) Prepare the tooth and abutment build-up material.

\*Note\* Auto-polymerization time (in oral cavity)  
 - In using mixing tip : within 4 min  
 - In using eco-tip (hand mixing) : within 6 min

**■ Characteristic**

Compressive strength	310 MPa
Polymerization depth(20 sec)	5.1mm

**■ Package**

- ChemiCore 5mL(10g) syringe x 2 ea, Eco tip, Mixing tip

**■ Cautions**

1. Operation time is accelerated or delayed by about 5 sec depending on 1°C rising/falling.
2. This product is not removed when being stained on the clothes, pay attention to avoiding staining of it.
3. When your skin or eyes come in contact with this product, rinse them with clean water several times and seek medical advice if needed.
4. Do not use a product having passed its expiry date.
5. Do not use or touch this product other than dental practitioners.

**■ Storage**

1. Avoid hot temperature and strong sunrays.
2. Close the cover immediately after use and store it.
3. Store this product in a dry and cool place.
4. This product is valid for 2 years after its manufacturing date.

**■ Side effects**

In rare cases, ChemiCore may cause a sensitizing reaction for hypersensitive patients due to any of the ingredients of these materials. In these cases, ChemiCore must no longer be used.

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