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20200829 SDM-007 (Rev.2)

Manufacturer	
Authorized Representative In the European Community	
CE (Notified Body)	
Batch Code	
Date of Manufacture	
Use By	
Non Sterile	
Do not use if package is damaged	
Consult Instructions for use	
Temperature Limitation	
Keep dry	
Keep away from sunlight	
Caution	
Do not re-use	

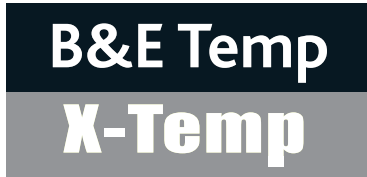
Manufacturer
B&E KOREA Co., Ltd.
 #807, #802, B Dong, 60, Haan-ro,
 Gwangmyeong-si, Gyeonggi-do, Korea
 Tel. 82-70-7425-2818 Fax. 82-2-6112-2816

EC Representative
BIOGERI
 Am Pflaushof 12, 22393 Hamburg, Germany
 Tel. +49-40-4143-102150 Fax. +49-40-4143-102159

B&E KOREA CO., LTD.
 B&E KOREA 1루비엔이코리아
 www.bnekorea.com

Made in Korea

Instruction for Use

B&E Temp / X-Temp is a light-cured temporary filling materials. It has the characteristics of easy removal from cavities, high elasticity, and great strength. Until a long-term restorative material can be used to fill cavity, it has the role for maintaining the shape of a tooth.

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B&E Temp / X-Temp

B&E Temp / X-Temp is a light-cured temporary filling materials. It has the characteristics of easy removal from cavities, high elasticity, and great strength. Until a long-term restorative material can be used to fill cavity, it has the role for maintaining the shape of a tooth.

Application range

B&E Temp / X-Temp is applicable for the temporary filling of a variety of cavities, it is especially suitable for Inlay and Onlay.

Application method

1. Before the Application
 - 1) Check the expiration date and avoid package contamination.
 - 2) Place a disposable tip on the syringe.
2. Application
 - 1) Prepare the cavity by cleaning and drying.
 - 2) Apply the filling material to the cavities..
 - 3) Check the occlusion between B&E Temp / X-Temp and the cavity, and then remove excess of material.
 - 4) Light-curing the material for 10-20 seconds.

Package

1.2ml x 4 syringes, disposable tips
 3g x 3 syringes

Cautions

1. B&E Temp / X-Temp is only used for oral tissue.
2. Avoid contact with eyes.
3. In case of contact with the skin or eyes, clean with pure water several times, and then consult with a medical specialist if necessary.
4. Do not use expired B&E Temp / X-Temp
5. Use by trained and qualified dentist
6. For dental use only

Storage

1. Do not store it under conditions of high temperature or intense light.
2. Close the cap immediately after use.
3. Store in dry and cool place.
4. Valid for 2 years from the manufacturing date.

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