20200829 SDM-021 (Rev.3)

Manufacturer	
Authorized Representative In the European Community	
CE (Notified Body)	
Batch Code	
Date of Manufacture	
Use By	
Non Sterile	
Do not use if package is damaged	
Consult Instructions for use	
Temperature Limitation	
Keep dry	
Keep away from sunlight	
Caution	
Do not re-use	

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**B&E Etch is phosphoric acid(H3PO4) semi-gel type agent intended for use on dentain and enamel.**

**B&E KOREA CO., LTD.**

**B&E Etch is phosphoric acid(H3PO4) semi-gel type agent intended for use on dentin and enamel. B&E Etch will effectively remove the smear layer so it helps successful bonding.**

**Indication for use**  
Pre-treatment agent which helps to bonding agent having a strong bonding strength resin and teeth.

**Contraindication**  
If the stipulated working technique cannot be used, the materials should not be applied.

**Applicaton methdod**

1. Isolation & Drying
  - Clean and dry the cavity
  - Avoid and contamination of the cavity with saliva or blood
  - Rubber dam is recommended for isolation
2. Cavity preparation
  - Reparation cavity with minimum elimination of the teech
3. Etching
  - Etch entire cavity with B&E Etch 10-30 sec
4. Rinse for 15 sec by water

**Package** 5 mL x 3 syringe, tips

**Cautions**

1. B&E Etch is corrosive and contains phosphoric acid
2. Avoid contact with eyes
3. In case of contact with skin or eyes, clean with pure water several times, and the consult with a medical specialist if necessary
4. Do not use expired B&E Etch
5. For dental use only
6. Reuse of the tip may lead to infection

**Storage**

1. Do not store it under conditions of high temperature or intense light
2. Close the cap immediately after use
3. Store in dry and cool place
4. Valid for 3 years from the manufacturing date

**Side effect**  
In rare cases, B&E Etch may cause a sensitizing reaction for hypersensitive patients due to any of the ingredients of these materials. In these cases, B&E Etch must no longer be used.

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