

RETREATY

Endodontics kit

a specific kit designed to perform endodontic
disassembling in a minimally invasive way
following Styleitaliano Endodontics philosophy.

Power by: 




RETREATY

Endodontics kit

Nowadays retreatment therapies are increasing compare to primary treatment that's why we decided to create a specific kit to perform endodontic disassembling in a minimal invasive way following Styleitaliano Endodontics philosophy.

Power by:  STYLEITALIANO

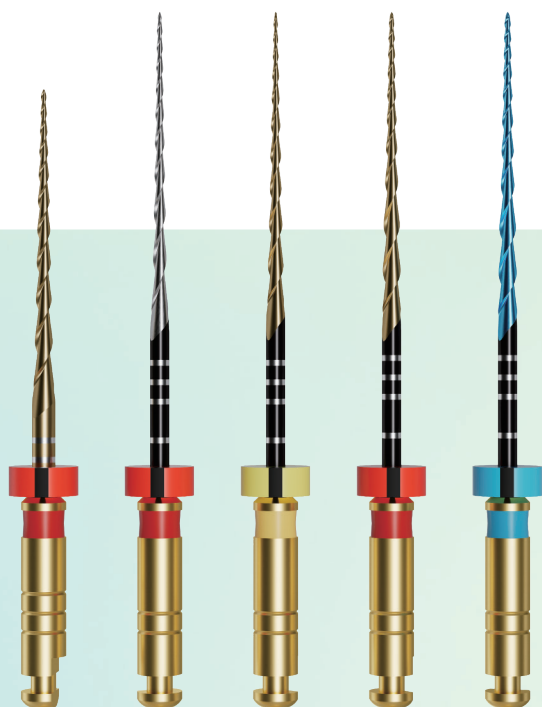

Size	Taper	Torque
Bull #25/.07	07	1.5N•CM
SkinnY #25/.04	04	1.5N•CM
ShapY 1 #20/.05	05	1.5N•CM
ShapY 2 #25/.05	05	1.5N•CM
ShapY 3 #30/.05	05	1.5N•CM

Cross section: 

Speed range: 350~500 RPM



BULL

Has been specifically designed to remove the coronal Gutta-Percha obturation material during a non-surgical endodontic retreatment. The gold treatment of the alloy, the tip design, and the shorter length of the file provide to the instrument the right features for removing in a short time the Gutta-Percha without any solvent.



SKINN^Y

Thin, but powerful for debris removal, silver color, used for removing Gutta-Percha in the middle apical third of the root canal system preserving the coronal region of the canal and creating a reproducible pathway close to the apical terminus.



SHAP^Y 1

Probably the most trendy deep shape that could be achieved in a retreatment. The rotary file has a gold alloy for a good flexibility combined to an unwinding resistance.



SHAP^Y 2

It's the first real shaper after that the patency has been reached. Gold alloy, flexible but with a great cutting efficacy, designed to follow the natural anatomy of the canal.



SHAP^Y 3

Blu alloy for higher flexibility incase of apical shape enlargement. Maximum adaptation without alteration of the original anatomy.